

## Technical data ESA712 Electrical Safety Analyzer

# Our easiest-to-use electrical safety analyzer, ever.

The ESA712 Electrical Safety Analyzer is a gamechanger in manual electrical safety testing. This manually operated, portable device delivers unparalleled versatility and precision. Catering to the needs of healthcare technology management professionals, the ESA712 performs from basic troubleshooting to comprehensive analysis on medical equipment, in the field and in facilities. Meeting global electrical safety standards, this innovative device combines the functions of a safety analyzer and patient simulator into one, offering a complete, manual solution for your electrical safety testing needs.

#### **Key features**

- 5" Touchscreen with intuitive user interface
- Small, portable, ergonomic design with an integrated handle
- Ten applied parts posts with innovative LED lighting guidance (ECG & Banana)
- Insulation resistance from 50V, 100V, 250V and 500V
- Compliance to key electrical safety standards NFPA 99 / AAMI ES1, AS/NZS 3551, IEC 62353 and parts of EN 50678 / EN 50699, and IEC 60601-1
- ECG waveform tests and dual-lead measurements combine the functionality of a simulator and safety analyzer in a single test tool
- Easy data entry through barcode scanner, external keyboard or on-screen keyboard



- 20 A at 120 V equipment current capability
- Replaceable mains fuses keep your unit in the field and out of the repair shop
- Rigorously tested for rugged field applications with CE and CSA in addition to Fluke-quality ruggedness for longterm reliability
- Enjoy precision, compliance, and total peace of mind through our comprehensive service and coverage program, eliminating unexpected costs with our Protect+ Program
- Global support network delivering prompt service and peace of mind to Fluke Biomedical customers worldwide

RaySafe™ XICTOBEEN



General Specifications		
Safety standard compliance	IEC 61010-1: Overvoltage category II, pollution degree 2 IEC 61010-2-034: Measurement CAT II 300 V	
Measurement equipment standard compliance	IEC 61557-16:2014, except IP40 for certain equipment outlets	
Dimensions (w x d x h)	214 x 207 x 92 mm (8.4 x 8.1 x 3.6 inches)	
Weight	1.3 kg (2.7 pounds)	
Ingress protection	IP40 per IEC 60529, excluding equipment outlet	
Operating temperature	0 to +35 °C (+50 to +95 °F)	
Operating humidity	10-90 %, non-condensing	
Storage temperature	-20 to +60 °C (-4 to+ 140 °F)	
Storage humidity	5-95 %, non-condensing	
Battery charging temperature	erature 8 to +28°C (+46 to +82°F)	
Altitude	100-127 V ac mains voltage and ≤5 150 V on input jacks: ≤ 5000 m	
	200-240 V ac mains voltage and ≤5 300 V on input jacks: ≤ 2000 m	
Battery	Built-in rechargeable lithium-ion (<3600 mAh, 3.6 V)	
Battery life	Up to 2 h	
Connectivity	1 x USB-C for PC communication, 2 x USB-A for peripherals	
Display	5-inch touchscreen	
Data storage	> 10 000 measurements	
Power (region dependent)	90-132 V ac, 20 A MAX, 47-63 Hz	
	90-132 V ac & 180-264 Vac, 10 A MAX, 47-63 Hz	
	90-132 V ac & 180-264 V ac, 15 A MAX, 47-63 Hz	
	90-132 V ac & 180-264 V ac, 16 A MAX, 47-63 Hz	
CSA approval		
US version, 90-132 V ac, 20 A MAX, 47-63 Hz	CSA approved	
NEMA 6-15 version, 90-132 V ac & 180-264 V ac, 15 A MAX, 47-63 Hz	CSA approved	
Electromagnetic Compatibility (EMC)		

See users manual for more information.





#### **General Specifications (continued)**

Measurement Specifications		
Mains voltage		
Range	0-264 V ac rms, limited by power specification	
Accuracy	±(2 % + 0.2 V)	
Point to point voltage		
Range	0-300 V ac rms, limited by altitude specification	
Accuracy	±(2 % + 0.2 V)	
Protective earth resistance & Point	to point resistance	
Range	0-20 Ω	
Accuracy	±(1 % + 0.01 Ω) at ≤ 2 Ω	
	±(1 % + 0.1 Ω) at >2 Ω	
Test current	Square wave of at least $\pm 200$ mA for $\leq 2 \Omega$	
Open circuit voltage	Max ±24 V	
Equipment Current		
Range	0-20 A ac rms, limited by power specification	
Accuracy	±(5 % + 0.05 A)	
Max duty cycle	0-10 A: continuous	
	10-15 A: 7 min on/3 min off	
	15-20 A: 5 min on/5 min off	
Note: Equipment outlet will be turned off	if max duty cycle is exceeded.	
Leakage Current		
Modes	ac + dc (true rms), ac only, dc only Mains on applied part leakage test: ac only	
Patient load	AAMI ES1:1993 IEC 60601-1:2005 / IEC 62353:2014 See users manual for more information.	
Crest factor	<2	
Range	0 μA - 20 mA	
Accuracy	dc & 20 Hz - 0.5 kHz: ±(1 % + 1 μA)	
	0.5 kHz - 50 kHz: ±(2.5 % + 1 μA)	
	50 kHz - 1 MHz: ±(5 % + 1 μA)	
Mains on applied part (Applies to: mains equipment leakage)	on applied part leakage, direct applied part leakage, alternative applied part leakage, alternative	

Test voltage	Mains voltage ±5 %	
Current limit	1 mA ± 25 % at 115 V for AAMI ES1, NFPA 99 3.5 mA ± 25 % at 230 V for IEC 60601-1, IEC 62353 and EN 50678 / EN 50699 7.5 mA ± 25 % at 230 V for AS/NZS 3551	
Additional uncertainty	Up to $\pm 2~\mu A$ for 120 V, up to $\pm 4~\mu A$ for 230 V	

**Note:** For alternative equipment leakage, alternative applied part leakage, direct equipment leakage, and direct applied part leakage tests, the leakage is scaled to nominal mains as per IEC 62353. Specified accuracy is not valid if (unscaled) current limit is exceeded.



## Measurement Specifications (continued)

Insulation Resistance			
Test voltage 250 V dc and 500 V dc	Range	0.1-100 ΜΩ	
	Accuracy	±(2 % + 0.2 MΩ) at ≤ 10 MΩ ±(7.5 % + 0.2 MΩ) at >10 MΩ	
	Test voltage accuracy	+20 % / -0 %	
Test voltage 50 V dc and 100 V dc	Range	0.1-20 ΜΩ	
	Accuracy	±(10 % + 0.2 MΩ)	
	Test voltage accuracy	+30 % / -0 %	
Short-circuit current	2 mA ± 0.25 mA		
Maximum load capacitance	2 µF		

ECG simulation			
Frequency accuracy	±2 %		
Amplitude accuracy	±5 %, for 2 Hz square wave		
Waveforms	ECG complex	30, 60, 120, 180 and 240 BPM	
	Square wave	0.125 and 2 Hz, 50 % duty cycle	
	Triangle wave	2 Hz	
	Pulse, 63 ms	30 and 60 BPM	
	Ventricular fibrillation		
Respiration simulation			
Rate	Apnea (0 BrPM) and 10-100 BrPM in 10 BrPM steps		
Waveforms	Normal		
Inspiration : expiration ratio	1:1		
Impedance baseline	1000 $\Omega \pm 5$ % between leads		
Impedance variation	1 ± 0.15 Ω		
Respiration lead	LL or LA, user selectable		



### Ordering information

#### Models/descriptions

6046003	ESA712 Electrical Safety Analyzer-US, 120 V
6046042	ESA712 Electrical Safety Analyzer-01-France, 230 V
6045879	ESA712 Electrical Safety Analyzer-02-Europe, 230 V
6046035	ESA712 Electrical Safety Analyzer-03-Isreal, 230 V
6046019	ESA712 Electrical Safety Analyzer-04-Denmark, 230 V
6045867	ESA712 Electrical Safety Analyzer-05-Australia, 230 V
6045898	ESA712 Electrical Safety Analyzer-06-UK, 230 V
6045908	ESA712 Electrical Safety Analyzer-07-Australia, 230 V
6046026	ESA712 Electrical Safety Analyzer-08-Thailand, 230 V
6046192	ESA712 Electrical Safety Analyzer-09-Japan, 100 V
6046163	ESA712 Electrical Safety Analyzer-10-North America, 220 V
6046205	ESA712 Electrical Safety Analyzer-11-Brazil, 230 V
6046057	ESA712 Electrical Safety Analyzer-12-India, 230 V
6045996	ESA712 Electrical Safety Analyzer-13-South Africa, 230 V
6046171	ESA712 Electrical Safety Analyzer-14-NEMA615

#### **Standard accessories**

5517611	Getting started guide
5517610	OneQA quick guide
5579600	USB C to USB A cable
2427138	Probe test banana jack 4MM tip black w/cap
	175-290-001 EVZFTP74SW00 (USA/AUS/ISR/THAI/JAPAN)
2099044	Probe test banana jack 4MM tip red w/cap
	175-290-003 EVZFTP74RT00 (USA/AUS/ISR/THAI/JAPAN)
4151242	USA/NEMA outlet to NBR14136 socket (BRAZIL)
3326842	Null post adapter
2248650	Carrying case

LINE CORD Country-specific power cord

#### Accessory kits (country specific)

3111008	USA/AUS/ISR Accessory Kit: test lead set, TP1 test probe set,
	AC285 alligator clip set (ESA T/L kit, USA)
3111024	EUR Accessory Kit: test lead set, TP74 test probe set,
	AC285 alligator clip set (ESA T/L kit, EUR)

## Optional accessories

6078258	Barcode	scanner

6078184 USB Wi-Fi dongle



#### About Fluke Biomedical

Fluke Biomedical is the world's leading manufacturer of quality biomedical test and simulation products. In addition, Fluke Biomedical provides the latest medical imaging and oncology quality-assurance solutions for regulatory compliance. Highly credentialed and equipped with a NVLAP Lab Code 200566-0 accredited laboratory, Fluke Biomedical also offers the best in quality and customer service for all your equipment calibration needs.

Today, biomedical personnel must meet the increasing regulatory pressures, higher quality standards, and rapid technological growth, while performing their work faster and more efficiently than ever. Fluke Biomedical provides a diverse range of software and hardware tools to meet today's challenges.

# Fluke Biomedical regulatory commitment

As a medical test device manufacturer, we recognize and follow certain quality standards and certifications when developing our products. We are ISO 9001 and ISO 13485 medical device certified and our products are:

- CE Certified, where required
- NIST Traceable and Calibrated
- UL, CSA, ETL Certified, where required
- NRC Compliant, where required



#### Fluke Biomedical

We empower our everyday heroes to focus only on protecting lives.

Fluke Biomedical

6920 Seaway Boulevard Everett, WA 98203

For more information, contact us at:

(800) 850-4608 or Fax (440) 349-2307 sales@flukebiomedical.com flukebiomedical.com

©2025 Fluke Biomedical. Specifications subject to change without notice. 5/2025 22955a-en

Modification of this document is not permitted without written permission from Fluke Corporation.

FLUKE

RaySafe<sup>\*</sup>